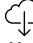



Updating SHOEBOX Transducers Guide

Before updating your transducers it's important that you have the most recent version of SHOEBOX installed. You can check by going to the App Store and making sure no new updates are available.

Once you've verified that you're up to date, and you have received your newly calibrated transducer(s), follow the steps below to update the associated calibration files.

- 1 If you're not already logged in to your SHOEBOX Audiometer, do this now.
- 2 On the home page, tap "Settings." Then select "Transducers" and delete all transducers you are no longer using from the enabled and disabled transducer lists.
- 3 Click the cloud icon  to "scan barcode" or "enter code manually."

Your new transducer(s) will now be listed in the "Active Transducers" section.

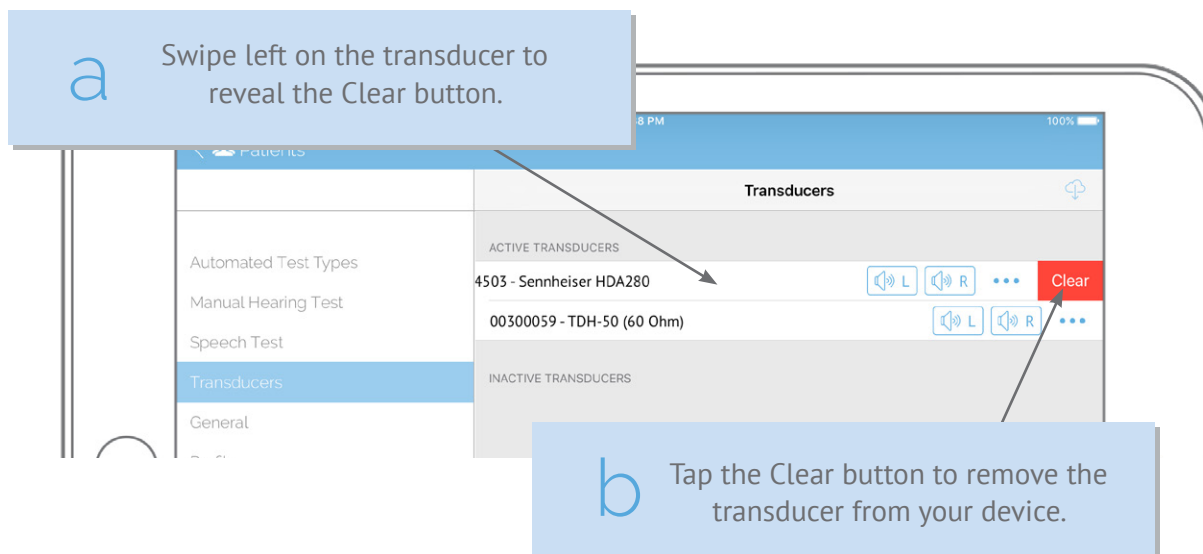

SHOEBOX is manufactured by
Clearwater Clinical Ltd.
301 - 80 Aberdeen St
Ottawa, ON, Canada K1S 5R5
www.clearwaterclinical.com
1-877-349-9934

SB-QMS-723 Rev H / 2018-08